



J DENTAL CARE

just smile

JD EXTRA DRILLS KIT

Guided Surgery Kit



JD Extra Drills Kit for guided surgery

The JD Guided Surgery Extra Drills Kit is a kit that includes drills to be used for JDNasal, JDPterygo JDIcon Plus PT and JDIcon Plus LE implant placement using the guided surgery technique.

Drills:

JDGD20-180	Guided Drill Ø 2.0 L 18.0
JDGD20-200	Guided Drill Ø 2.0 L 20.0
JDGD20-220	Guided Drill Ø 2.0 L 22.0
JDGD20-240	Guided Drill Ø 2.0 L 24.0
JDGD20-260	Guided Drill Ø 2.0 L 26.0
JDGD24-180	Guided Drill Ø 2.4 L 18.0
JDGD24-200	Guided Drill Ø 2.4 L 20.0
JDGD24-220	Guided Drill Ø 2.4 L 22.0
JDGD24-240	Guided Drill Ø 2.4 L 24.0
JDGD24-260	Guided Drill Ø 2.4 L 26.0



JD Extra Drills Kit:

JDKIT02	JD Guided Surgery Kit Extra Drills
---------	------------------------------------



Drilling Protocols

Pterygomaxillary surgical protocol

IMPLANT DIAMETER	IMPLANT LENGTH	
Ø 3,3	L13	2,0 L6 - L8 - L10 - L11,5 - L13 2,4 L6 2,8 L6 (only for dense bone)
	L15	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 2,4 L6 2,8 L6 (only for dense bone)
	L18	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 2,4 L6 2,8 L6 (only for dense bone)
	L20	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 2,4 L6 2,8 L6 (only for dense bone)
Ø 4,0	L13	2,4 L6 - L8 - L10 - L11,5 - L13 3,2 L6
	L15	2,4 L6 - L8 - L10 - L11,5 - L13 - L15 3,2 L6
	L18	2,4 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 3,2 L6
	L20	2,4 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 3,2 L6

Note: all measurements in mm

Trans-sinus surgical protocol

IMPLANT DIAMETER	IMPLANT LENGTH	
Ø 4,0	L20	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 2,4 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 2,8 L6 - L8 - L10 - L11,5 - L13 3,2 L6 - L8 - L10 3,6 L6
	L22	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 - L22 2,4 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 - L22 2,8 L6 - L8 - L10 - L11,5 - L13 3,2 L6 - L8 - L10 3,6 L6
	L24	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 - L22 - L24 2,4 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 - L22 - L24 2,8 L6 - L8 - L10 - L11,5 - L13 3,2 L6 - L8 - L10 3,6 L6
	L26	2,0 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 - L22 - L24 - L26 2,4 L6 - L8 - L10 - L11,5 - L13 - L15 - L18 - L20 - L22 - L24 - L26 2,8 L6 - L8 - L10 - L11,5 - L13 3,2 L6 - L8 - L10 3,6 L6

